# DENTOCROWN HD

TEMPORARY CROWNS & BRIDGES

ITENA®

## FR

### DESCRIPTION :

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### INDICATIONS :

- Fabrication de temporary crown, bridges, inlays, onlays, partial crown and veneers.

### APPLICATION:

Insérer la cartouche de Dentocrown HD dans un distributeur avec un piston adéquat. Due to technical reasons, slight differences in the filling level between the cartridge chambers can occur. In this case, before use, remove the cartridge cap and squeeze the material out until the material is delivered evenly from both chambers. Then insert a mixing tip and lock with a 90° turn clockwise. By processing the lever the material is extruded and automatically mixed in the correct ratio. After use, do not remove the mixing tip from the cartridge. Replace the used mixing tip with a new one just before the next application. After use, place the cap back onto the cartridge.

### PROCESSING DIRECTIONS:

Level out any undercuts in the preparation using a non-eugenol cement.

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2. Dry the teeth and lightly lubricate the area by applying a lubricant such as Vaseline o similar materials.

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4. Gentle insert into the mouth and firmly hold the prepared tooth (within its impression material) for up to 45 seconds.

5. Approximately 1:30 to 1:45 minutes from start of mix, the provisional crown or bridge can easily be removed together with the impression from the prepared teeth. The setting reaction has to be monitored intraorally, as the temperature in the mouth has a significant influence on the setting time and removal of the provisional is only possible during gel state.

6. The crown and bridge may be contoured and polished after 2:30 additional minutes. The removal of the soft, sticky inhibition layer on the surface of Dentocrown HD (e.g. by wiping with ethyl alcohol) facilitates the shaping. Completed crown can be finished using toothed grinders, etc.

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### STORAGE:

- Do not expose restorative materials to elevated temperature or intense light.
- Store this material at a temperature between 2-27°C.
- Ambient temperature routinely higher than 27°C/80°F may reduce shelf life.

5. See outer cartridge for expiration date.

6. Do not store materials in proximity to products containing Eugenol.

### WARRANTY:

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the suitability of the materials delivered by us for the intended purposes of application. As the application of our preparation is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

### CUSTOMER INFORMATION:

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

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