EN Instructions for Use

- Etching Gel;
37% phosphoric acid

DE Gebrauchsinformation

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Instructions for Use

Description

Total Etch is a gel used for enamel etching and dentin conditioning in conjunction with sealing and direct composite restorations as well as adhesive cementation of lab-fabricated all-ceramic or composite restorations.

Composition

Total Etch contains phosphoric acid (37 wt.% in water), thickening agent and colour pigments.

Indication

Total Etch is applied using the enamel etch or total etch technique for the following indications:

- Composite restorations
- Adhesive cementation of inlays (composite, all-ceramic), crowns, bridges, veneers, adhesive bridges and root canal posts
- Splints
- Bonding of orthodontic attachments (e.g. brackets)
- Sealing of fissures

Contraindication

If the stipulated application technique cannot be applied.

Side effects

None known to date.

Other interactions

Glass-ionomer-based liners may be dissolved by Total Etch.

Application

Apply Total Etch on enamel and subsequently on dentin; and allow a reaction time of 15 seconds. Afterwards, thoroughly rinse off the etchant with water spray and dry the tooth surfaces with oil-free air. Avoid excessive drying of the dentin. The etched enamel surface should have a chalky white appearance. If this is not the case, repeat the etching procedure using a shorter reaction time. If the tooth surface becomes accidentally contaminated (e.g. with

CE 0123

Rx ONLY
Date information prepared:
2014-04-25/Rev. 3
602997/WE3

 **Manufacturer**
Ivoclar Vivadent AG
9494 Schaan/Liechtenstein
www.ivoclarvivadent.com

ivoclar
vivadent®
clinical

saliva), repeat the etching procedure with a reaction time of max. 10 seconds.

The reaction time on unprepared enamel (e.g. fissure sealants) is 30–60 seconds. If the enamel is selectively etched, use a reaction time of 15–30 seconds.

Refilling empty Total Etch syringes

Empty 2-gram syringes can be conveniently refilled with the contents of the Total Etch Jumbo syringe up to 15 times.

Important

Although the 2-gram syringes are equipped with a plunger safety stop, it is absolutely necessary to wear gloves, protective goggles and a face mask for safety reasons.

- Remove the screw-on cap from the 30-gram Jumbo syringe and the empty 2-gram syringe.
- Join the tips of the two syringes using a twisting, clockwise motion. Make sure that the two syringes are firmly connected. Note: Do not strip the thread.
- Refill the small syringe by slowly and steadily pulling back the plunger of the small syringe. Simultaneously exert light pressure on the plunger of the jumbo syringe using the other hand to support the refilling process.
- **Avoid overfilling the small syringe.**
- **Under no circumstances should you exceed the plunger safety stop of the small syringe due to the risk of acid burn!**
- Do not point the syringes at people during refilling.
- Do not fill etchant from the 2-gram syringe back into the Jumbo syringe.
- Disconnect the two syringes using a counter-clockwise twist.
- Replace the screw-on cap on the Jumbo syringe.
- The syringe is now ready for use. Attach an application tip onto the 2-gram syringe. Apply the etchant as described above.
- Use only the Total Etch application tips delivered with this package. The use of other application tips may have an adverse effect on the viscosity of the etchant.
- In order to ensure traceability, only the enclosed 2-gram syringes should be refilled with the contents of the Jumbo syringe delivered with the same package.

- Please indicate the LOT number on the Jumbo syringe in case of any query.

Warning

When filling the 2-gram syringe, always wear protective goggles and gloves.

Total Etch contains phosphoric acid and is corrosive. Avoid contact with the eyes, mucous membrane, and skin (goggles are recommended for both patients and operators). After contact with the eyes, immediately rinse with copious amounts of water for at least 15 minutes and see a physician/opthalmologist. After skin contact, immediately rinse with soap and water. After contact with sensitive materials (surfaces, clothing), immediately clean with copious amounts of water.

Storage

- Do not use Total Etch after the date of expiration.
- Storage temperature: 2–28 °C (36–82 °F). Recommendation: Refrigerate large inventory at 2–8 °C (36–46 °F).
- Date of expiration: see note on syringe/packaging.

Keep out of the reach of children.

For use in dentistry only.

This material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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Ivoclar Vivadent Limited
Ivoclar Vivadent, Inc.