


TECHNICAL FILE**CE: - Class: -
GMDN: -**

PRODUCT NAME	PRODUCT NUMBER
Dentorama Tray Cleaner	CHE-080
PRODUCT DESCRIPTION	
Tray cleaner for quick cleaning of impression trays in aluminium or stainless steel. Powder to be diluted in hot water. Bottle of 1000 g.	
Do not mix aluminium and stainless steel impression trays in the same solution. Aluminium impression trays should neither be mixed with chromium plated or nickel-based impression trays.	
No CE-marking required.	
DIRECTIONS FOR USE	
Used by professionals in dentistry only.	
Mix 5 tbs. of powder in 1 litre of hot water. Leave the trays in the solution for a few minutes and then rinse. Diluted powder (used or unused) is recommended to be kept not more than 1 day.	
Irritant to eyes and skin, mark with:	
	

SEPARATE BROCHURE AVAILABLE



MATERIAL SPECIFICATION	
Sodium carbonate, calcinated, light	30-60%
Sodium hexameta phosphate	30-60%
STERILISATION	
STORAGE CONDITIONS	
Store in original container tightly closed and in a dry area. Temperature between 8 °C – 28 °C. Store separate from food, forage, fertilizer, etc.	
STANDARDS / TESTS PERFORMED	

Latest update: WJ 2015-05-13