

GB	IMPORTANT - READ CAREFULLY	021 / 04
-----------	-----------------------------------	-----------------



ALUMINIUM PRODUCT

- **DO NOT USE IN A DISHWASHER OR WASHER DISINFECTOR.**
- **DO NOT USE IN AN ULTRASONIC BATH.**
- **BE CAREFUL WHEN USING DISINFECTING LIQUIDS: some are incompatible with aluminium: Please pay special attention to the instructions enclosed with, or on, the product you intend to use.**

This product is manufactured from coloured anodised aluminium (even the silver / grey colour). The colour of the product can be damaged by contact with some chemical cleaners.

DO NOT USE chemicals that are too alkaline or too acid, a Ph of between 4 and 8 is safe.

ALWAYS READ the manufacturers instructions of any product prior to it's use on aluminium products.

THE MANUFACTURER will not accept responsibility for the improper use of chemical cleaners resulting in damage and discolouration of any product.

PLEASE NOTE: the use of Ultrasonic cleaning baths can result in the loss of the Anodised colour. Any product damaged due to the improper use of cleaning / disinfecting solutions, or ultrasonic baths will not be exchanged.

FOR STERILIZATION the following processes can be used safely: autoclave (134°), chemiclave (132°), or dry heat (180°). Please refer to the manufacturer's instructions for the correct cycle and or time.