Technical Data Sheet

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*Set for machinery irrigation system to be used with dispensing liquid coolants in oral surgery*

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*Products are classified in Class IIa in accordance with Rule 5 of Annex IX to Directive 93 /42/EEC and subsequent 2007/47/EC and are CE marked 0546.*

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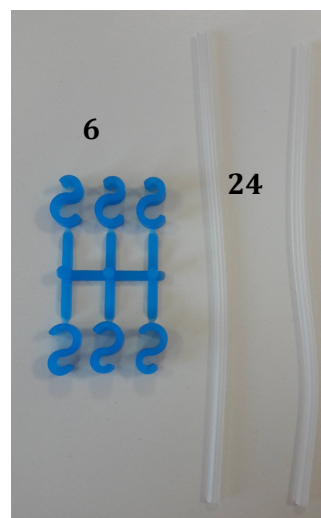
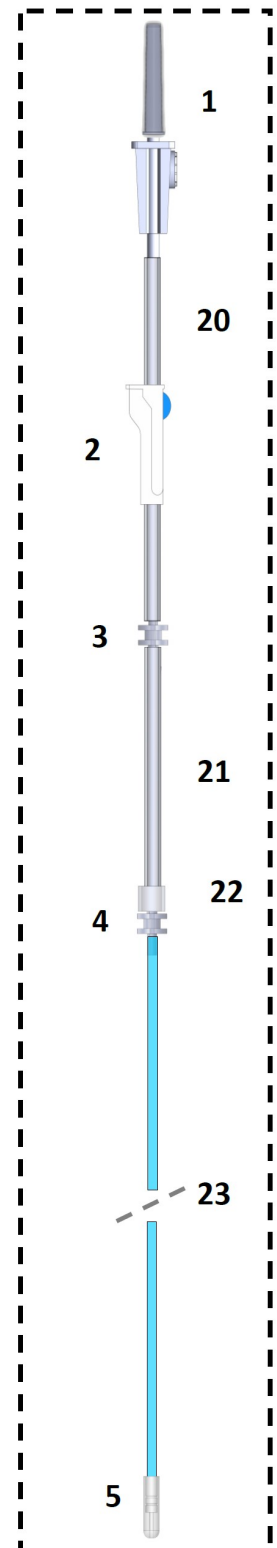
*Products are packaged in specific envelope labeled with all information about code, lot, expiry date, manufacturer and CE mark.*

<b>CONF. PRIMARIO FIRST PACKAGING</b>	<b>CONF. SECONDARIO SECONDARY PACKAGING</b>	<b>CONF. ESTERNO EXTERNAL PACKAGING</b>
<b>1 PC/ 1PC</b>	<b>10 PZ /10 Pcs</b>	<b>60 PZ /60 Pcs</b>
Carta medica 60 gr/m <sup>2</sup> – Film plastico PP50/PE30 PEEL Dimensioni del blister: 25x11,6 cm	Scatola 20x28x12 cm	Scatola 37,5x28x41 cm
<i>Medical paper 60 gr/m<sup>2</sup> - Plastic film PP50/PE30 PEEL Blister dimensions: 25x11,6 cm</i>	<i>Box 20x28x12 cm</i>	<i>Box 37,5x28x41 cm</i>

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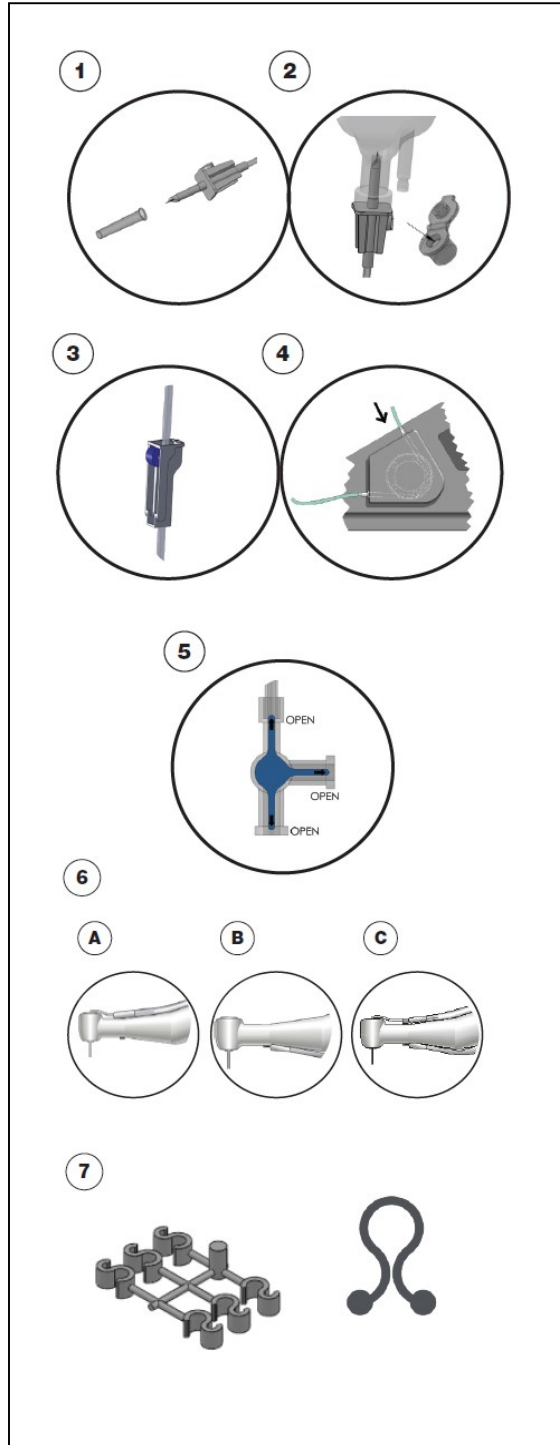
**GB**

Before the use, make sure that the product is sterile. The product is intended to be sterile when the packaging is intact and doesn't show any damage. If the blister is intact, remove the irrigation set from the sterile packaging.

1. Remove the cap from the perforator.
2. Insert the perforator into the bag or connect the irrigation system via Luer Lock. The perforator has an air filter with an open-close cap. Keep the filter open if using glass or rigid plastic bottles.
3. Where foreseen, adjust the flow with the roller.
4. Insert the hose in the peristaltic pump, check the flow direction and use the appropriate connection at the end of the silicon hose.
5. Connect the final hose to the three ways tap and regulate the flow direction.
6. Connect the final part of the irrigation hose to the contra-angle.

Omnia irrigation sets offer the following options:

- a. Single way internal irrigation
  - b. Single way external irrigation
  - c. Double way internal and external irrigation, by means of an Y connection supplied in the packaging
7. Fix the irrigation hose to the cord of the micro motor using the clips included: S tube holders for Physiodispenser and heparin hook for piezosurgery.
  8. If you want to use a manual sprinkler, connect it to the three-way tap.
  9. Prime the line and check the patency of the fittings and the tightness of the line itself. In the event of a leak, check the source of the leak and replace the entire system if necessary.

- Hose set for machinery driven irrigation system, latex free, sterilized with ethylene oxide, is a single use product. Do not sterilize again, do not reuse.
- Do not store close to direct heating sources, keep in a dry place.
- Dispose the irrigation set following the local rules.

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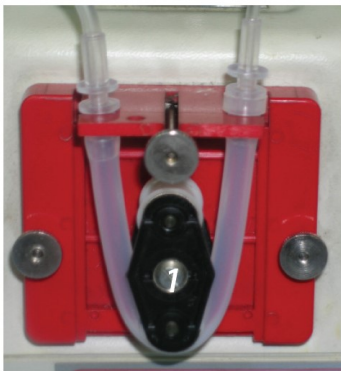
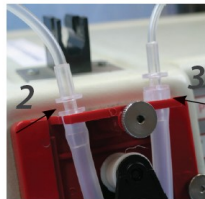
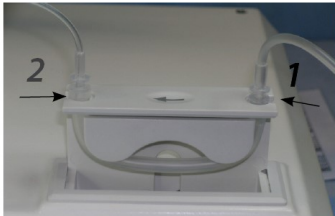
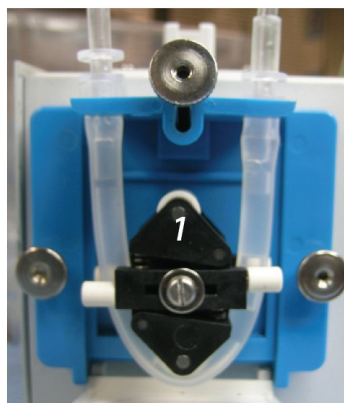
*Product sterilization until opening the envelope*

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Sterilization method: *ethylene oxide*

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*Validity: 5 years from date of sterilization*

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